

**ENGLISH CONTENT**  
\* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

\* hermetically sealed aluminium pouch containing:  
1 KIDNEYS ALBUMIN TEST cassette, 1 desiccant bag  
Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked. The desiccant packet must not be used. Dispose of it with household waste without opening it.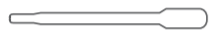

\* pipette

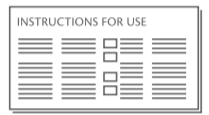\* 1 instructions for use leaflet

**Material required but not supplied:**

- a device to measure time (i.e. timer, watch)
- clean, dry, residue-free container.

**TEST PROCEDURE**

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry.
- 2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. – **FIG. A**
- 3) Collect urine in a clean clean, dry, residue-free container (not supplied) – **FIG. B**
- 4) Press the pipette bulb and dip the pipette tip into the collected urine. Release the bulb to collect the urine sample. – **FIG. C**
- 5) Gently press the pipette bulb and deposit 3 drops of sample into the well (S) indicated on the cassette, wait 1-2 seconds between dispensing one drop and the next. – **FIG. D**
- 6) Wait 5 minutes and read the results as follows.

**RESULTS INTERPRETATION**

**READ THE RESULT AFTER EXACTLY 5 MINUTES**  
The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

**NEGATIVE RESULT**

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means that the levels of albumin in the urine are normal or lower than the value used to calibrate the test (10 µg/ml).

**POSITIVE RESULTS**

A coloured band appears only under the C (control) sign. This result means the levels of albumin in the urine are higher than normal.

**NON VALID RESULT**

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test.

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