Distributed by:  
**berkeleyhealth**  
52 Berkeley Square  
Mayfair London W1J 5BT UK  
+44 20 8191 0500  
support@berkeleyhealth.com  
www.berkeleyhealth.com

**CE** **REF** Qarad EC-REP BV  
Pas 257  
2440 Geel - Belgium

**PRIMA LAB SA**  
Via Antonio Monti 7  
CH-6828 Balerna - SWITZERLAND  
+41 91 605 1030  
support@primalabs.ch  
primalabs.ch

**REF** 890092  
890092\_IFU\_72\_5.0 08/2022

Registered trademarks.

## berkeleyhealth TETANUS

### TETANUS PREVENTION

- Self-test for the determination of the level of antibodies (class G Immunoglobulins – IgG) against tetanus in whole blood samples
- Selbsttest zur Bestimmung der Antikörperspiegel (Immunglobuline der Klasse G – IgG) in Vollblutproben
- Auto-test pour la détermination des niveaux d'anticorps (immunoglobuline de classe G – IgG) antitétaniques dans les échantillons de sang entier

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### ENGLISH TETANUS AND VACCINE COVERAGE

Tetanus is a non-contagious, infective disease caused by the *Clostridium tetani* bacterium. This bacterium is often present as a spore in the soil and can penetrate into the human body via contaminated scratches and wounds. Once inside the body, the spore transforms into the vegetative form, which produces a very powerful neurotoxin. The toxin spreads around the body through the blood and the lymphatic system, causing spasms and widespread contractions. Without suitable treatment, the disease can be fatal in 30-50% of cases. Vaccination is the only way to prevent the disease. The administration of three doses of vaccine provides very high protection, with effectiveness above 95%. The duration of the protection is approximately 10 years and it is further guaranteed by periodical boosters. TETANUS TEST is a qualitative, immunological self-test to identify antibodies against the tetanus toxin in whole blood. It reveals if the user has antibody protection against infection.

**Warning: Do not use in emergency situations or suspected of being infected with Clostridium tetani (wounds and/or injuries with rusty iron objects, handling soil, hay, animal excrement and during gardening activities es puncture rose thorn). In these cases, it is recommended to contact a doctor.**

### PRINCIPLE OF THE TEST

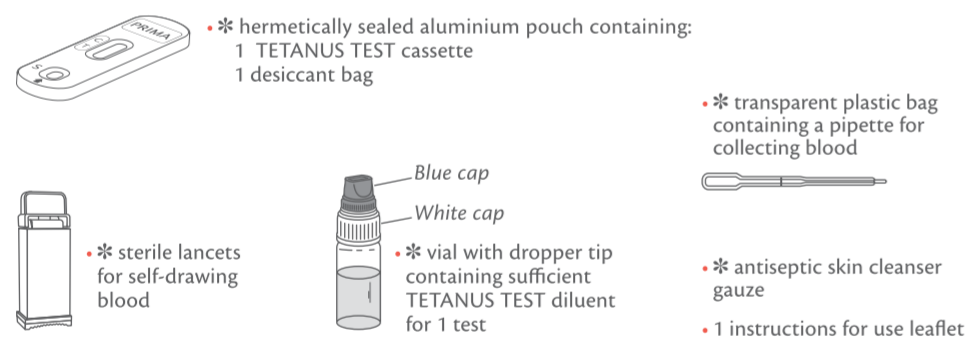
TETANUS TEST is an immunochromatographic test that detects the presence in the blood of IgG antibodies against the tetanus toxin, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

### REFERENCES

- Tetanus vaccines: WHO position paper – February 2017.
- Bae C, Bourget D.: "Tetanus." StatPearls Publishing; 2018 Jan.
- WHO Immunological Basis for Immunization Series Module 3: Tetanus Update 2018.

### CONTENT

\* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.


- \* hermetically sealed aluminium pouch containing: 1 TETANUS TEST cassette 1 desiccant bag

\* transparent plastic bag containing a pipette for collecting blood

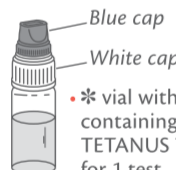- \* sterile lancets for self-drawing blood

- \* vial with dropper tip containing sufficient TETANUS TEST diluent for 1 test

\* antiseptic skin cleanser gauze

- 1 instructions for use leaflet

\* Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).

\* Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.

\* The desiccant packet must not be used. Dispose of it with household waste without opening it.

### PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the procedure exactly, using only the specified quantities of blood and diluent.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test and lancing device once only.
- The test is for external use only. DO NOT SWALLOW.
- In vitro diagnostic device for individual use.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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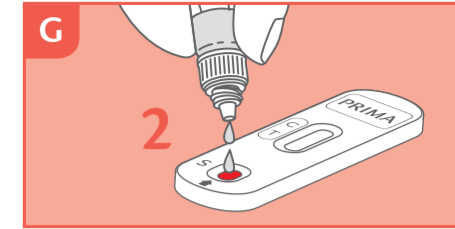
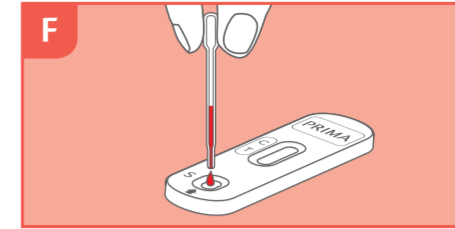
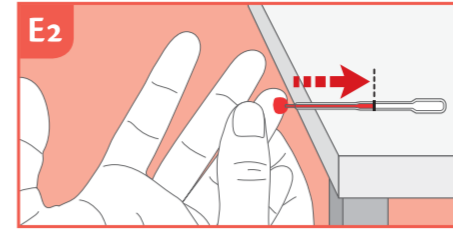
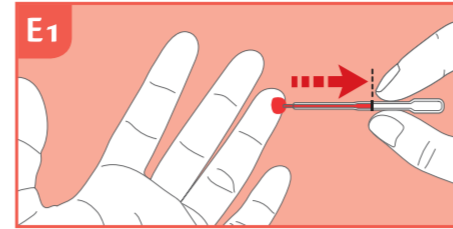
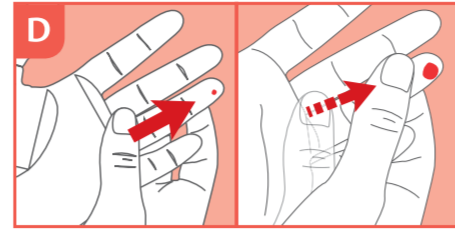
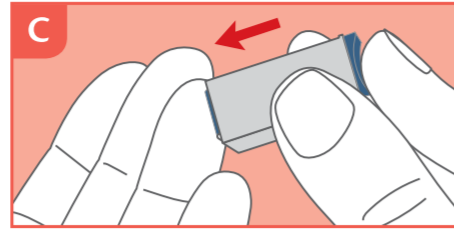
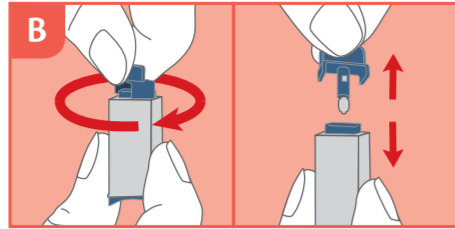
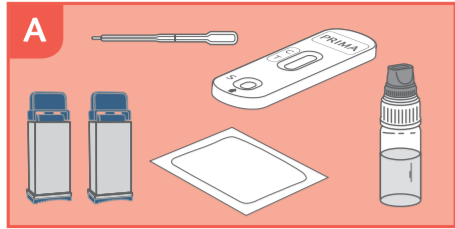
- TEST PROCEDURE**
- Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.  
*Note: The use of warm water facilitates capillary blood collection as it induces vasodilation.*
  - Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag. Open the plastic packet containing the pipette. – **FIG. A**
  - Carefully rotate the protective cap of the sterile lancet 360° without pulling it. Extract and discard the released cap. – **FIG. B**
  - Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended). It is important that the massage is done from the palm of the hand to the phalanx, to improve blood flow. Press the open end of the lancet (the side the cap has been extracted from), against the fingertip – **FIG. C**. The tip of the lancet automatically retracts after use. If the lancet does not work properly, discard it and use the second one supplied. If the second one is not required, it can be disposed of without special precautions.
  - Holding the hand down, massage the finger until a large drop of blood forms. It is important to massage from the palm of the hand to the phalanx to improve blood flow. – **FIG. D**
  - Take the pipette without pressing the bulb. Two sampling methods are suggested:  
– **FIG. E1:** hold the pipette horizontally without pressing the bulb place it in contact with the drop of blood, it will enter the pipette by capillarity. Move the pipette away when the black line is reached. If there is not enough blood, continue to massage the finger until the black line is reached.  
– **FIG. E2:** place the pipette on a clean, flat surface with the tip protruding from the shelf, then place the drop of blood in contact with the pipette, it will enter by capillarity. If blood is not sufficient, continue massaging the finger until the blood has reached the black line.  
Avoid, as far as possible, to continuously move the tip of the pipette away from the finger in order to prevent the formation of air bubbles.
  - Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb. – **FIG. F**
  - Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on). Deposit 2 drops into the well indicated on the cassette (S), wait for 5 seconds between the first and second drop. – **FIG. G**
  - Wait 5 minutes and read the results referring to the next section for result interpretation.

**RESULTS INTERPRETATION**  
**READ THE RESULTS AFTER 5 MINUTES (NOT OVER 6 MINUTES)**

**POSITIVE RESULT**  
Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means that the levels of anti-tetanus antibodies in the blood are equal to or higher than 0.1 IU/ml and therefore you are covered against tetanus infection.

**WEAK POSITIVE RESULTS**  
Positive results can be displayed (as very weak test lines) even at IgG concentrations of tetanus antitoxin between 0.05 and 0.099 IU/mL. These concentrations can give low protection against *Clostridium tetani* and a vaccination booster is required. It has been observed, in a small percentage of cases, that the test can give weak positive test line also in the cut-off concentration range between 0.1 and 0.2 IU/mL. In case of weak positive result, it is recommended to consult your doctor for more information.

**NEGATIVE RESULTS**  
A coloured band appears only under the C (control) sign. This result means that the value of anti-tetanus antibodies is below 0.1 IU/ml and therefore you have no immunological cover against tetanus. Consult a physician.

**NON VALID RESULT**  
If no bands appear or if there is only a line under the T (Test) sign, it means that it was not possible to read the result of the test, which must be considered as not valid. It is recommended to repeat the test with a new blood sample.

**FAQ. – QUESTIONS AND ANSWERS**

- HOW DOES THE TETANUS TEST WORK?** A positive result from the test means that you are immunologically protected against tetanus because the detected concentration of IgG against the tetanus toxin in the blood is above 0.1 IU/ml. The Test has been calibrated on the International Standard NIBSC TE-3.
- WHEN CAN THE TEST BE USED?** The TETANUS TEST can be carried out at any time of the day. The immunity protection is usually effective for a period of ten years. The test can be used in normal conditions to see if you are still effectively protected a few years after the vaccination. Do not use in emergency situations or suspected of being infected with *Clostridium tetani* (wounds and/or injuries with rusty iron objects, handling soil, hay, animal excrement and during gardening activities as puncture rose thorn).
- DOES THE TETANUS TEST MEASURE OTHER PARAMETERS?** No. The TETANUS TEST only measures the immune cover against tetanus (suitable concentration of anti-tetanus antibodies in human blood). The test does not reveal if there is an ongoing *Clostridium tetani* infection.
- CAN THE RESULT BE INCORRECT?** The result is correct if the instructions are followed carefully. However, the result may not be correct if: the device comes into contact with other liquids before being used, if part of the diluent is accidentally spilt, if the amount of blood is not enough or the number of drops dispensed in the well is more than 3, or if the indicated reading time is not respected. The plastic pipette supplied allows users to be sure they have collected the right amount of blood.
- HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE**

**DIFFERENT?** The color and the intensity of the bands are important for the interpretation of the result. The bands must be full and homogeneous. To evaluate the intensity of the test line (positive – weak positive), refer to "result interpretation" section and pictures.

**IS THE RESULT RELIABLE IF READ AFTER 5 MINUTES?** No. The test must be read 5 minutes after the procedure is completed and no later than 6 minutes from this time. Results read after 6 minutes can be incorrect (possibility to obtain false positive result).

**WHAT SHOULD I DO IF THE RESULT IS POSITIVE?** A positive result means that the concentration of anti-tetanus antibodies is high enough to guarantee protection against infection. Despite the reliability of the test, falsely positive results are possible, therefore in case of emergency (wounds and/or injuries with rusty iron objects, handling soil, hay, animal excrement and during gardening activities as puncture rose thorn) it is still suggested to consult a physician.

**Tetanus Test, as qualitative assay, may give weak positive test line (please refer to "result interpretation" section to evaluate the test line intensity). The weak positive test line can be associated at a low protection against *Clostridium tetani* and a vaccination booster is required. It is recommended to consult your doctor for more information.**

**WHAT SHOULD I DO IF THE RESULT IS NEGATIVE?** A negative result means there is no protection against infection. You should immediately contact your physician and your vaccination center for information on how to have a tetanus vaccination booster.

**HOW ACCURATE IS THE TETANUS TEST?** The Test is very accurate. Assessment reports show a concordance rate above 90% (CI 95%: 89.3 - 98.6) with reference methods.

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